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STEEL FRAME FIXED-HEIGHT TABLES MARTINI: 220156, MARTINI PLUS: 220158

Storage: dry, dark place, -20°+60°

Cleaning: it is generally recommended to clean the device after each use and to carry out the disinfection cycle periodically. To clean the device, wipe its parts with a cloth moistened with warm water. Do not use any agents that could damage the surface of the device through abrasion or overly aggressive treatment.

Operating temperature: 0° / +40°

Expected lifetime: it is estimated that the device can be used for 10 years.

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CLASSIFICATION AND USE

STEEL FRAME FIXED-HEIGHT TABLES MARTINI: 220156, MARTINI PLUS: 220158

Read the instructions and use the product as directed according to its function.

Do not use the product in a way other than its intended purpose.

INTENDED USE

Fixed height tables with metal structure for diagnosis and treatment.
Tables for physical examinations and massages.

TYPE OF PATIENT

Patients requiring a physical examination, potentially also involving various types of instruments, or for massages for rheumatic type diseases. Also suitable for use for sports medicine massages.

PLACE OF USE

Hospitals, clinics, outpatient clinics, medical studios in general, private facilities nursery.

OPERATORS QUALIFIED TO USE THE PRODUCT

All healthcare professionals. The device can be used both by nursing staff and staff who have been specially trained.

CONTRAINDICATIONS

For generic use, not suitable for environments in which corrosive or unconventional products are used; if used in a modified atmosphere, ask the manufacturer for authorisation.
If in any doubt as to the operating environment, contact the manufacturer.

OTHER INDICATIONS

The patient resting surface should be covered with a disposable sheet or reusable sheet sanitised each time. It can be sanitised if it should come into contact with blood.

Report all serious incidents that occur due to the device. Report the incident to the manufacturer and the competent authority

At the end of its life, the product must be disposed of in accordance with local regulations and taken to specific waste collection facilities.

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Sport-Tec GmbH