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**STEEL FRAME FIXED-HEIGHT TABLES MARTINI: 220156, MARTINI PLUS: 220158**

**Storage:** dry, dark place, -20°+60°

**Cleaning:** it is generally recommended to clean the device after each use and to carry out the disinfection cycle periodically. To clean the device, wipe its parts with a cloth moistened with warm water. Do not use any agents that could damage the surface of the device through abrasion or overly aggressive treatment.

**Operating temperature:** 0° / +40°

**Expected lifetime:** it is estimated that the device can be used for 10 years.

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## CLASSIFICATION AND USE

### STEEL FRAME FIXED-HEIGHT TABLES MARTINI: 220156, MARTINI PLUS: 220158

Read the instructions and use the product as directed according to its function.

Do not use the product in a way other than its intended purpose.

#### INTENDED USE

Fixed height tables with metal structure for diagnosis and treatment.  
Tables for physical examinations and massages.

#### TYPE OF PATIENT

Patients requiring a physical examination, potentially also involving various types of instruments, or for massages for rheumatic type diseases. Also suitable for use for sports medicine massages.

#### PLACE OF USE

Hospitals, clinics, outpatient clinics, medical studios in general, private facilities nursery.

#### OPERATORS QUALIFIED TO USE THE PRODUCT

All healthcare professionals. The device can be used both by nursing staff and staff who have been specially trained.

#### CONTRAINDICATIONS

For generic use, not suitable for environments in which corrosive or unconventional products are used; if used in a modified atmosphere, ask the manufacturer for authorisation.  
If in any doubt as to the operating environment, contact the manufacturer.

#### OTHER INDICATIONS

The patient resting surface should be covered with a disposable sheet or reusable sheet sanitised each time. It can be sanitised if it should come into contact with blood.

**Report all serious incidents that occur due to the device. Report the incident to the manufacturer and the competent authority**

**At the end of its life, the product must be disposed of in accordance with local regulations and taken to specific waste collection facilities.**

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**Sport-Tec GmbH**